


## Rhino Rescue

# NASAL AIRWAY(NPA)




**01** INSPECT nose and nasal passages for obstructions preventing insertion of NPA.

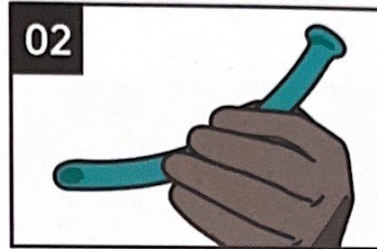

**02** OPEN the outer wrapper of the nasal airway.



**03** EXPOSE opening of casualty's right nostril by pushing the tip of the nose upward gently ("piggy the nose").

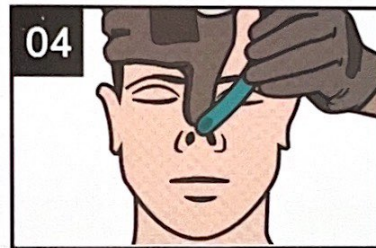

**04** POSITION tube so that the beveled (open) end faces toward the septum (the strip of skin separating the nostrils).



**05** INSERT NPA device into the right nostril (at a 90-degree angle to the casualty's face).



**06** POSITION casualty in either a sitting or recovery position.



**07** POSITION casualty in either a sitting or recovery position.